



Identification

Please check the reference number !

Turbines


BORA L	REF 1600382-001	PRESTIGE L	REF 1600380-001
BORA	REF 1600381-001	PRESTIGE	REF 1600379-001
BORALINA	REF 1600373-001	PRESTILINA	REF 1600372-001

Cleaning


SPRAYNET
Cleaning spray 500 ml,
box of 6 cans
REF 1600036-006



Outside

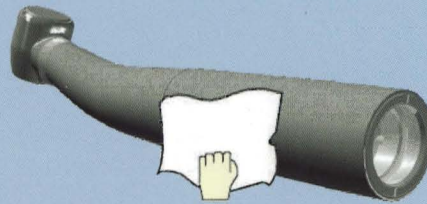

Inside

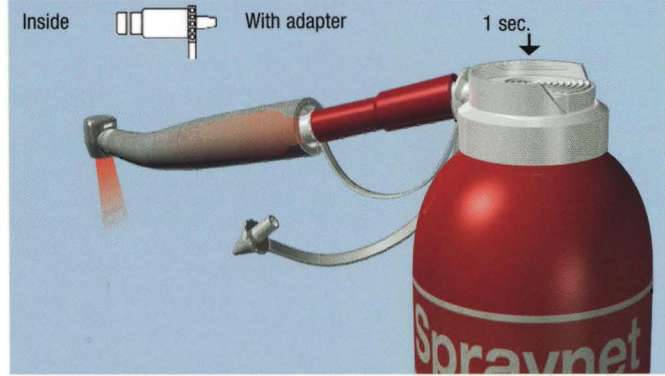


Lubrication

LUBRIMED
Medical grease cartridges,
box of 6 cartridges
REF 1600037-006


Lubrimed greaser
REF 1000003-001



Insert the tip of the cartridge in the chuck

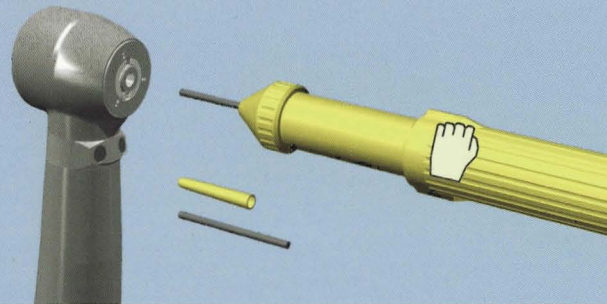

1/2 turn clockwise

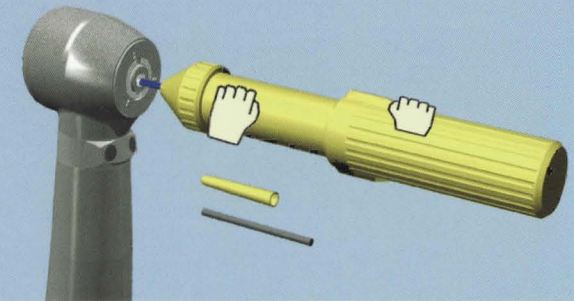


Sterilization

Steam autoclave



Before use, remove excess lubricant by running the attachment for 10 to 15 seconds at low speed

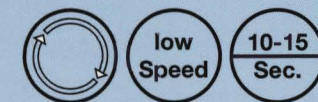

For further information, consult the operating instructions for the device

Maintenance animations at
www.bienair.com

BIEN-AIR USA, INC./IRVINE CA 92614
TEL. 1-800-433-BIEN/FAX 949-477-6051

Bien Air
SWITZERLAND
www.bienair.com